


Endovascular aortic repair – Fenestrated



Zenith®
ENDOVASCULAR GRAFTS




www.cookmedical.com/web/aortic-intervention/

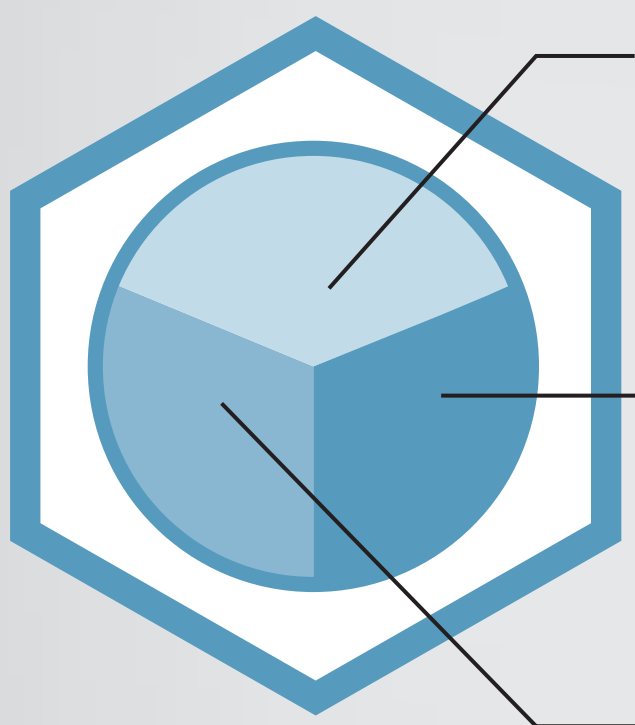


Active Fixation

Our anchoring barbs are the industry standard for design and migration resistance.

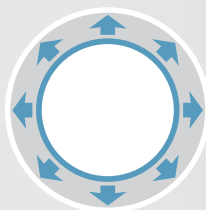

Radial Force

Self-expanding z-stents provide radial force, added stability and optimal graft-to-vessel apposition.

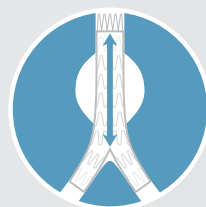

Columnar Strength

Long main-body design features a time-tested balance of length, stability and flexibility.

ARC Technology™ – Demand systemic migration resistance.

The three features that make up ARC Technology – active fixation, radial force and columnar strength – work in unison to give all Zenith® AAA endovascular grafts unrivaled migration resistance.

www.cookmedical.com/arc

In the U.S. Zenith® Fenestrated Clinical Study, there were two reports of migration, both in patients with evidence of disease progression at follow-up (without aneurysm pressurization).

Endovascular aortic repair – Fenestrated



Fenestrated4-14

Zenith® Fenestrated AAA Endovascular Graft

Overview 4-5

Graft specifications and ordering information 6-11

Zenith Spiral-Z® AAA Iliac Leg Graft


Graft specifications and ordering information 12-13

Zenith Alignment Stent

Ordering information 14

Accessories16-18



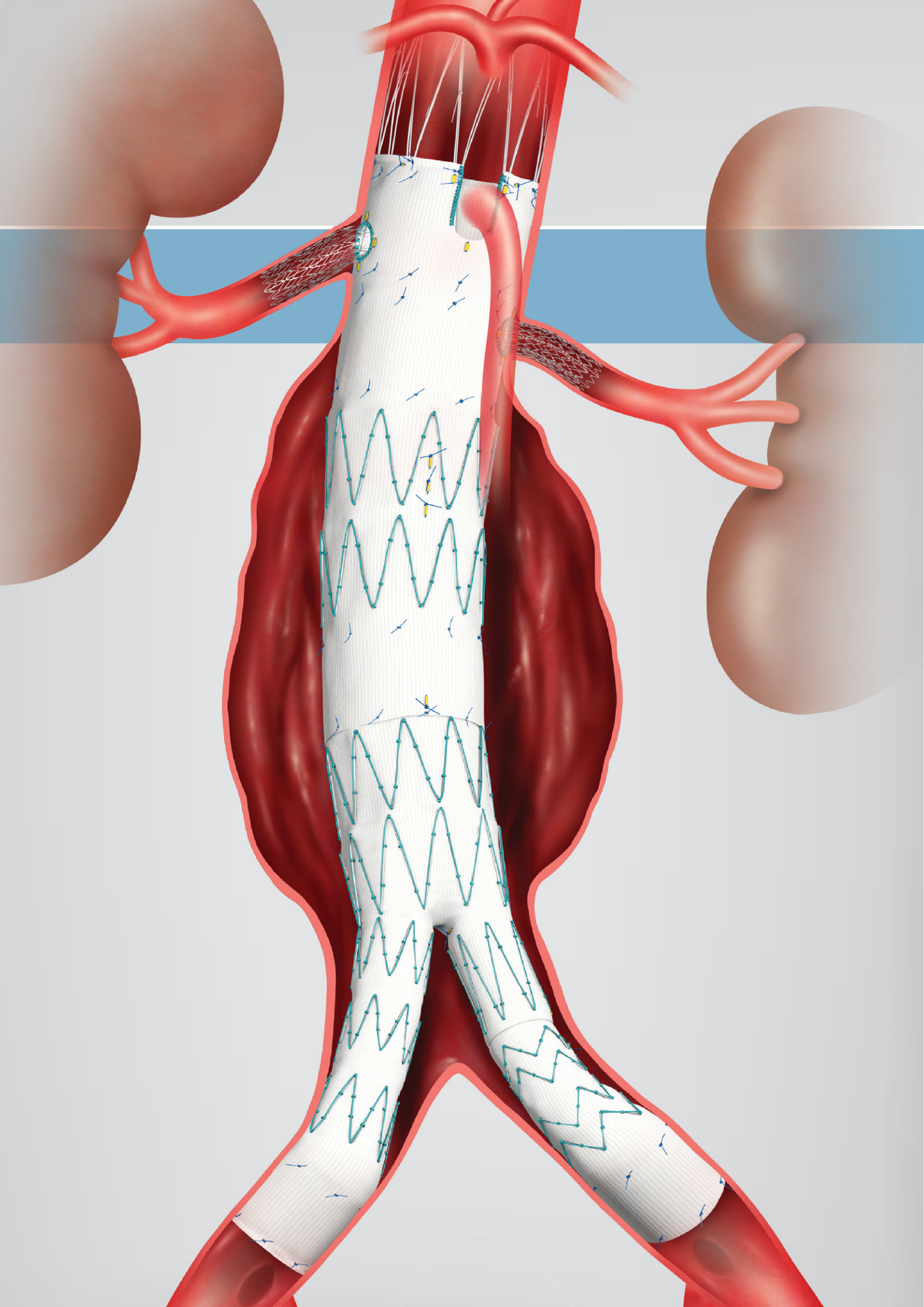

Experience patient-specific precision.

Zenith® Fenestrated

AAA ENDOVASCULAR GRAFT

Offer patient-specific endovascular solutions to more of your AAA patients. Zenith® Fenestrated is the first stent graft designed for patients with infrarenal necks as short as 4 mm.

Patient-specific precision

Achieve a seal just below the renal arteries without the risk of occluding any vessels thanks to precisely positioned fenestrations and scallops for the SMA and renal arteries.

- Scallops – 10 mm wide and 6-12 mm high
- Small fenestrations – 6 mm wide and 6 or 8 mm high
- Large fenestrations – 8-12 mm in diameter



scallop

small fenestration

large fenestration

Patient-specific modularity

Custom tailor Zenith Fenestrated further with our extensive selection of individual components, including the Zenith Spiral-Z® AAA Iliac Leg Graft. Zenith Spiral-Z provides even more treatment options for your patients with tortuous anatomy, since it has enhanced flexibility and is available in a wide range of lengths and diameters.

ARC Technology™

Experience systemic migration resistance due to the time-tested combination of active fixation, radial force, and columnar strength.


Precision delivery

H&L-B One-Shot™ introduction system

- Diameter-reducing ties facilitate placement and visceral vessel cannulation by constraining the graft diameter during deployment. This allows you to manipulate Zenith Fenestrated even when it's semi-deployed.
- Flexor® introducer sheath facilitates precise delivery and placement because of its kink-resistant technology and PTFE-coated lumen.
- AQ® hydrophilic coating facilitates precise delivery by dramatically reducing surface friction.
- Trigger wires facilitate staged deployment to ensure precise placement.
- Gold markers facilitate precise placement.

Proximal body grafts


One proximal internal stent

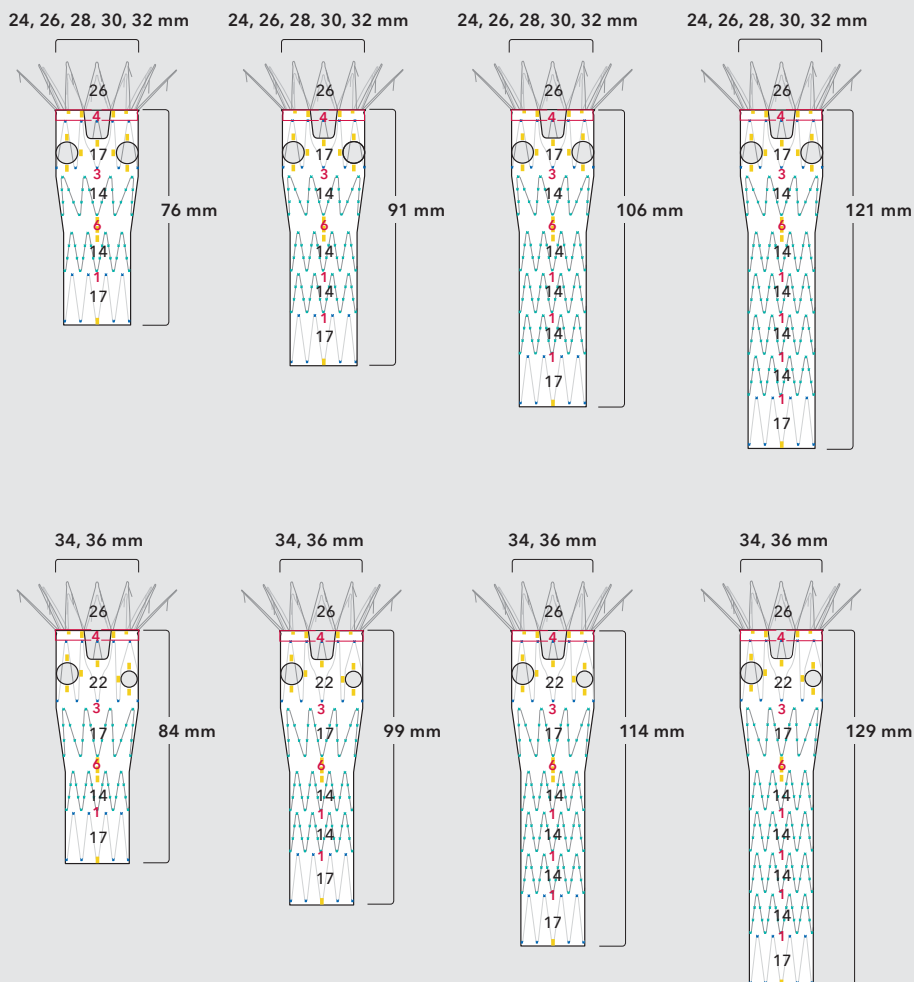



Note: Black numbers on stents indicate stent lengths in millimeters. Red numbers indicate gap lengths in millimeters. Fabric gaps may vary.

H&L-B One-Shot introduction system for proximal body grafts

24-34 mm grafts: 20 Fr (6.7) mm ID/7.7 mm OD
36 mm grafts: 22 Fr (7.3) mm ID/8.5 mm OD

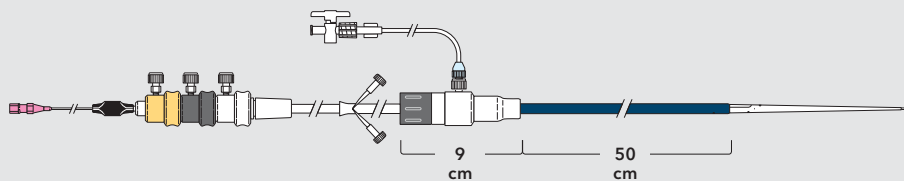

Zenith Fenestrated AAA - Component ordering information

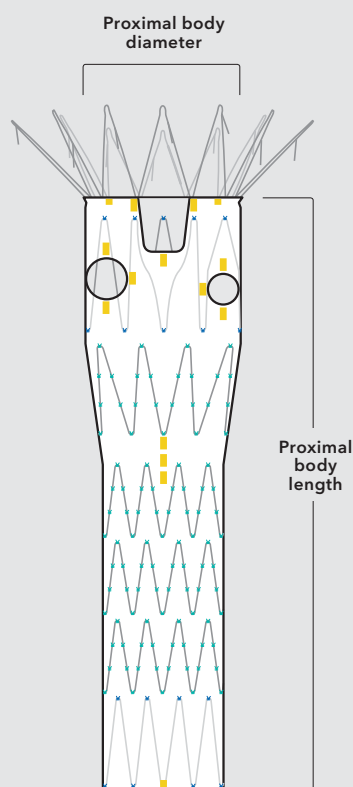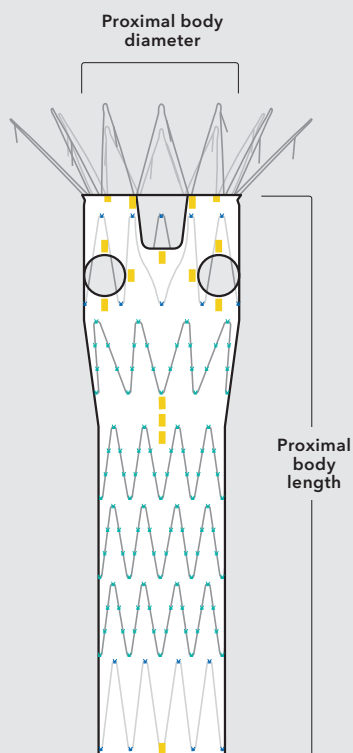


Order Number	Reference Part Number	Proximal Body Diameter mm	Proximal Body Length mm	Introducer Sheath ID/OD Fr (mm)/mm
One Proximal Internal Stent				
24 mm Diameter				
G32502	ZFEN-P-1-24-76-R	24	76	20 (6.7)/7.7
G32507	ZFEN-P-1-24-91-R	24	91	20 (6.7)/7.7
G32512	ZFEN-P-1-24-106-R	24	106	20 (6.7)/7.7
G32517	ZFEN-P-1-24-121-R	24	121	20 (6.7)/7.7
26 mm Diameter				
G32503	ZFEN-P-1-26-76-R	26	76	20 (6.7)/7.7
G32508	ZFEN-P-1-26-91-R	26	91	20 (6.7)/7.7
G32513	ZFEN-P-1-26-106-R	26	106	20 (6.7)/7.7
G32518	ZFEN-P-1-26-121-R	26	121	20 (6.7)/7.7
28 mm Diameter				
G32504	ZFEN-P-1-28-76-R	28	76	20 (6.7)/7.7
G32509	ZFEN-P-1-28-91-R	28	91	20 (6.7)/7.7
G32514	ZFEN-P-1-28-106-R	28	106	20 (6.7)/7.7
G32519	ZFEN-P-1-28-121-R	28	121	20 (6.7)/7.7
30 mm Diameter				
G32505	ZFEN-P-1-30-76-R	30	76	20 (6.7)/7.7
G32510	ZFEN-P-1-30-91-R	30	91	20 (6.7)/7.7
G32515	ZFEN-P-1-30-106-R	30	106	20 (6.7)/7.7
G32520	ZFEN-P-1-30-121-R	30	121	20 (6.7)/7.7
32 mm Diameter				
G32506	ZFEN-P-1-32-76-R	32	76	20 (6.7)/7.7
G32511	ZFEN-P-1-32-91-R	32	91	20 (6.7)/7.7
G32516	ZFEN-P-1-32-106-R	32	106	20 (6.7)/7.7
G32521	ZFEN-P-1-32-121-R	32	121	20 (6.7)/7.7
34 mm Diameter				
G32522	ZFEN-P-1-34-84-R	34	84	20 (6.7)/7.7
G32524	ZFEN-P-1-34-99-R	34	99	20 (6.7)/7.7
G32526	ZFEN-P-1-34-114-R	34	114	20 (6.7)/7.7
G32528	ZFEN-P-1-34-129-R	34	129	20 (6.7)/7.7
36 mm Diameter				
G32523	ZFEN-P-1-36-84-R	36	84	22 (7.3)/8.5
G32525	ZFEN-P-1-36-99-R	36	99	22 (7.3)/8.5
G32527	ZFEN-P-1-36-114-R	36	114	22 (7.3)/8.5
G32529	ZFEN-P-1-36-129-R	36	129	22 (7.3)/8.5

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Proximal body grafts


Two proximal internal stents

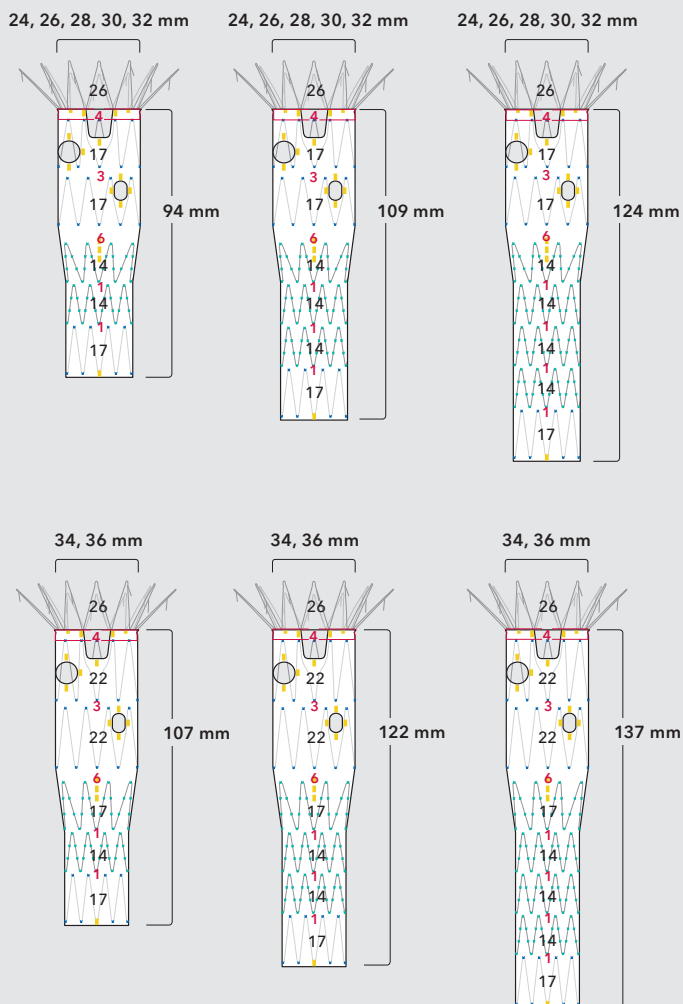


Note: Black numbers on stents indicate stent lengths in millimeters. Red numbers indicate gap lengths in millimeters. Fabric gaps may vary.

H&L-B One-Shot introduction system for proximal body grafts

24-34 mm grafts: 20 Fr (6.7) mm ID/7.7 mm OD
36 mm grafts: 22 Fr (7.3) mm ID/8.5 mm OD

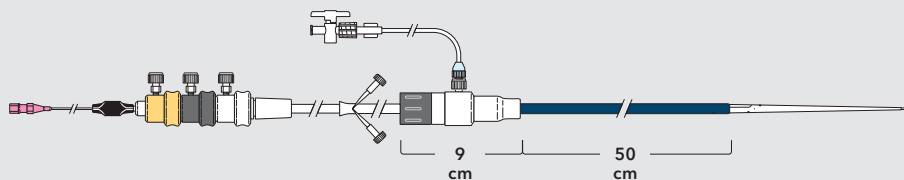


Zenith Fenestrated AAA - Component ordering information

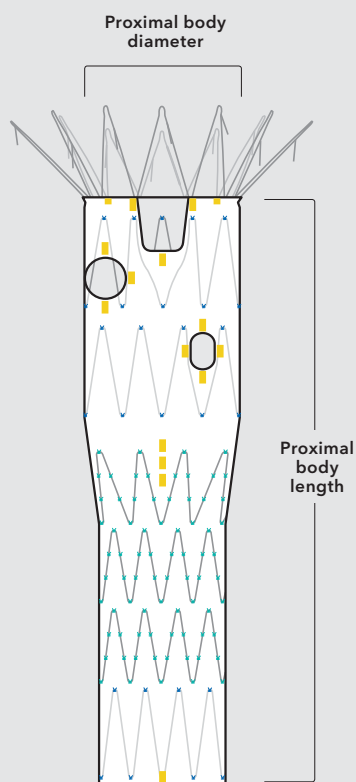

Order Number	Reference Part Number	Proximal Body Diameter mm	Proximal Body Length mm	Introducer Sheath ID/OD Fr (mm)/mm
Two Proximal Internal Stents				
24 mm Diameter				
G32530	ZFEN-P-2-24-94-R	24	94	20 (6.7)/7.7
G32535	ZFEN-P-2-24-109-R	24	109	20 (6.7)/7.7
G32540	ZFEN-P-2-24-124-R	24	124	20 (6.7)/7.7
26 mm Diameter				
G32531	ZFEN-P-2-26-94-R	26	94	20 (6.7)/7.7
G32536	ZFEN-P-2-26-109-R	26	109	20 (6.7)/7.7
G32541	ZFEN-P-2-26-124-R	26	124	20 (6.7)/7.7
28 mm Diameter				
G32532	ZFEN-P-2-28-94-R	28	94	20 (6.7)/7.7
G32537	ZFEN-P-2-28-109-R	28	109	20 (6.7)/7.7
G32542	ZFEN-P-2-28-124-R	28	124	20 (6.7)/7.7
30 mm Diameter				
G32533	ZFEN-P-2-30-94-R	30	94	20 (6.7)/7.7
G32538	ZFEN-P-2-30-109-R	30	109	20 (6.7)/7.7
G32543	ZFEN-P-2-30-124-R	30	124	20 (6.7)/7.7
32 mm Diameter				
G32534	ZFEN-P-2-32-94-R	32	94	20 (6.7)/7.7
G32539	ZFEN-P-2-32-109-R	32	109	20 (6.7)/7.7
G32544	ZFEN-P-2-32-124-R	32	124	20 (6.7)/7.7
34 mm Diameter				
G32545	ZFEN-P-2-34-107-R	34	107	20 (6.7)/7.7
G32547	ZFEN-P-2-34-122-R	34	122	20 (6.7)/7.7
G32549	ZFEN-P-2-34-137-R	34	137	20 (6.7)/7.7
36 mm Diameter				
G32546	ZFEN-P-2-36-107-R	36	107	22 (7.3)/8.5
G32548	ZFEN-P-2-36-122-R	36	122	22 (7.3)/8.5
G32550	ZFEN-P-2-36-137-R	36	137	22 (7.3)/8.5

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.


Distal bifurcated body grafts

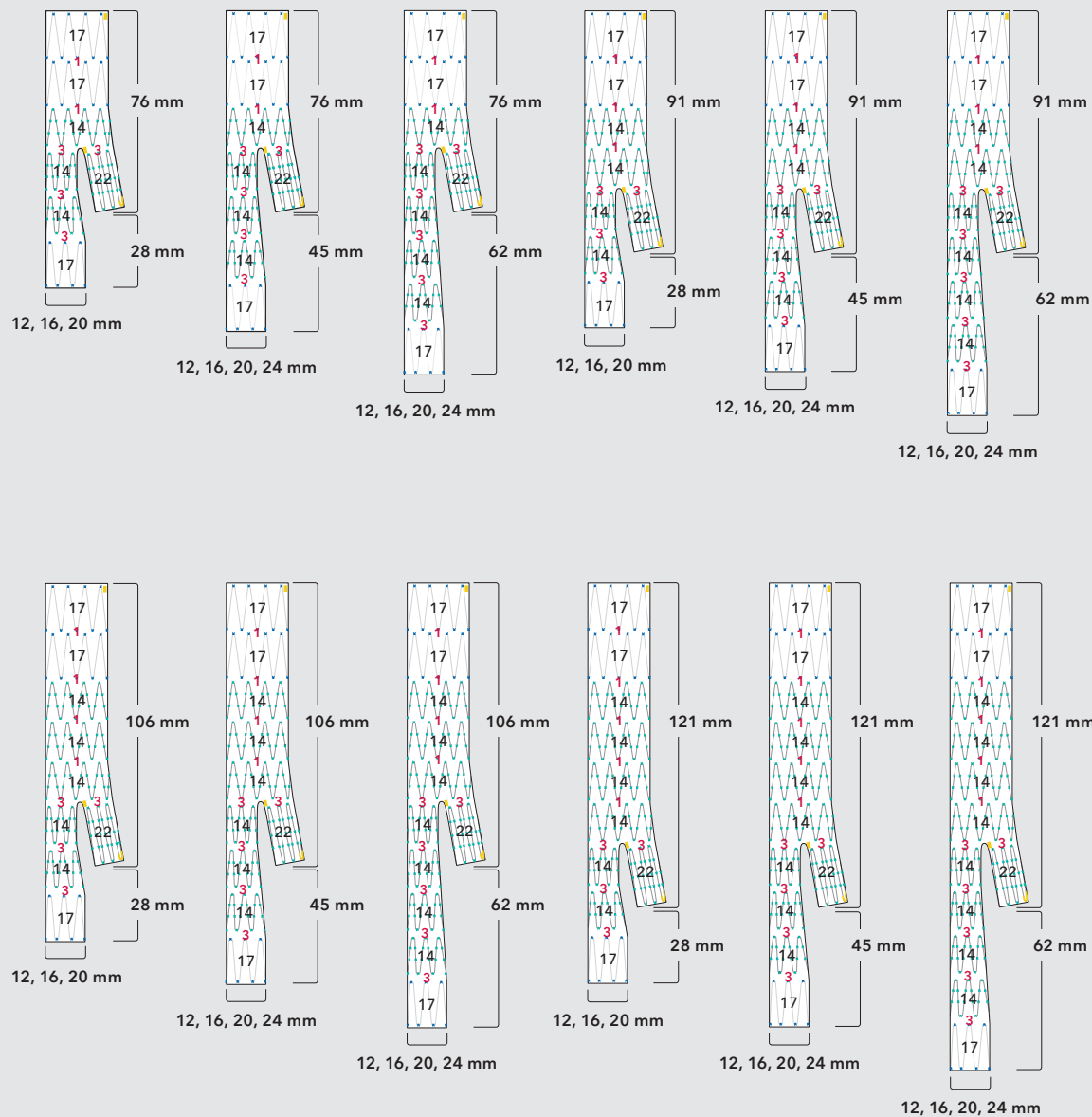


Note: Black numbers on stents indicate stent lengths in millimeters. Red numbers indicate gap lengths in millimeters. Fabric gaps may vary.

H&L-B One-Shot introduction system for distal bifurcated body grafts

20 Fr (6.7 mm) ID/7.7 mm OD

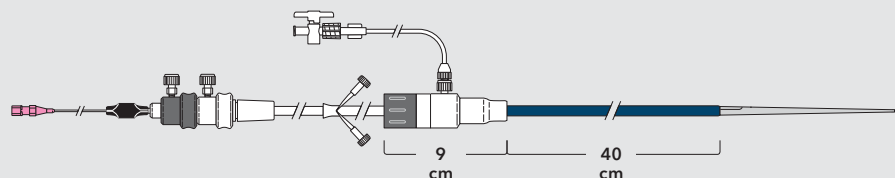


Zenith Fenestrated AAA - Component ordering information

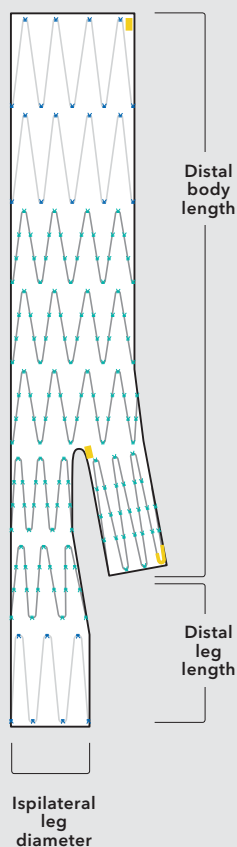

Order Number	Reference Part Number	Ipsilateral Leg Diameter mm	Distal Leg Length mm	Distal Body Length mm	Introducer Sheath ID/OD Fr (mm)/mm
Distal Bifurcated Body Graft					
G32551	ZFEN-D-12-28-76-C	12	28	76	20 (6.7)/7.7
G32554	ZFEN-D-12-28-91-C	12	28	91	20 (6.7)/7.7
G32500	ZFEN-D-12-28-106-C	12	28	106	20 (6.7)/7.7
G32501	ZFEN-D-12-28-121-C	12	28	121	20 (6.7)/7.7
G32561	ZFEN-D-12-45-76-C	12	45	76	20 (6.7)/7.7
G32565	ZFEN-D-12-45-91-C	12	45	91	20 (6.7)/7.7
G32569	ZFEN-D-12-45-106-C	12	45	106	20 (6.7)/7.7
G32573	ZFEN-D-12-45-121-C	12	45	121	20 (6.7)/7.7
G32577	ZFEN-D-12-62-76-C	12	62	76	20 (6.7)/7.7
G32580	ZFEN-D-12-62-91-C	12	62	91	20 (6.7)/7.7
G32584	ZFEN-D-12-62-106-C	12	62	106	20 (6.7)/7.7
G32588	ZFEN-D-12-62-121-C	12	62	121	20 (6.7)/7.7
G32552	ZFEN-D-16-28-76-C	16	28	76	20 (6.7)/7.7
G32555	ZFEN-D-16-28-91-C	16	28	91	20 (6.7)/7.7
G32557	ZFEN-D-16-28-106-C	16	28	106	20 (6.7)/7.7
G32559	ZFEN-D-16-28-121-C	16	28	121	20 (6.7)/7.7
G32562	ZFEN-D-16-45-76-C	16	45	76	20 (6.7)/7.7
G32566	ZFEN-D-16-45-91-C	16	45	91	20 (6.7)/7.7
G32570	ZFEN-D-16-45-106-C	16	45	106	20 (6.7)/7.7
G32574	ZFEN-D-16-45-121-C	16	45	121	20 (6.7)/7.7
G32592	ZFEN-D-16-62-76-C	16	62	76	20 (6.7)/7.7
G32581	ZFEN-D-16-62-91-C	16	62	91	20 (6.7)/7.7
G32585	ZFEN-D-16-62-106-C	16	62	106	20 (6.7)/7.7
G32589	ZFEN-D-16-62-121-C	16	62	121	20 (6.7)/7.7
G32553	ZFEN-D-20-28-76-C	20	28	76	20 (6.7)/7.7
G32556	ZFEN-D-20-28-91-C	20	28	91	20 (6.7)/7.7
G32558	ZFEN-D-20-28-106-C	20	28	106	20 (6.7)/7.7
G32560	ZFEN-D-20-28-121-C	20	28	121	20 (6.7)/7.7
G32563	ZFEN-D-20-45-76-C	20	45	76	20 (6.7)/7.7
G32567	ZFEN-D-20-45-91-C	20	45	91	20 (6.7)/7.7
G32571	ZFEN-D-20-45-106-C	20	45	106	20 (6.7)/7.7
G32575	ZFEN-D-20-45-121-C	20	45	121	20 (6.7)/7.7
G32578	ZFEN-D-20-62-76-C	20	62	76	20 (6.7)/7.7
G32582	ZFEN-D-20-62-91-C	20	62	91	20 (6.7)/7.7
G32586	ZFEN-D-20-62-106-C	20	62	106	20 (6.7)/7.7
G32590	ZFEN-D-20-62-121-C	20	62	121	20 (6.7)/7.7
G32564	ZFEN-D-24-45-76-C	24	45	76	20 (6.7)/7.7
G32568	ZFEN-D-24-45-91-C	24	45	91	20 (6.7)/7.7
G32572	ZFEN-D-24-45-106-C	24	45	106	20 (6.7)/7.7
G32576	ZFEN-D-24-45-121-C	24	45	121	20 (6.7)/7.7
G32579	ZFEN-D-24-62-76-C	24	62	76	20 (6.7)/7.7
G32583	ZFEN-D-24-62-91-C	24	62	91	20 (6.7)/7.7
G32587	ZFEN-D-24-62-106-C	24	62	106	20 (6.7)/7.7
G32591	ZFEN-D-24-62-121-C	24	62	121	20 (6.7)/7.7

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.


Contralateral/ipsilateral iliac leg grafts

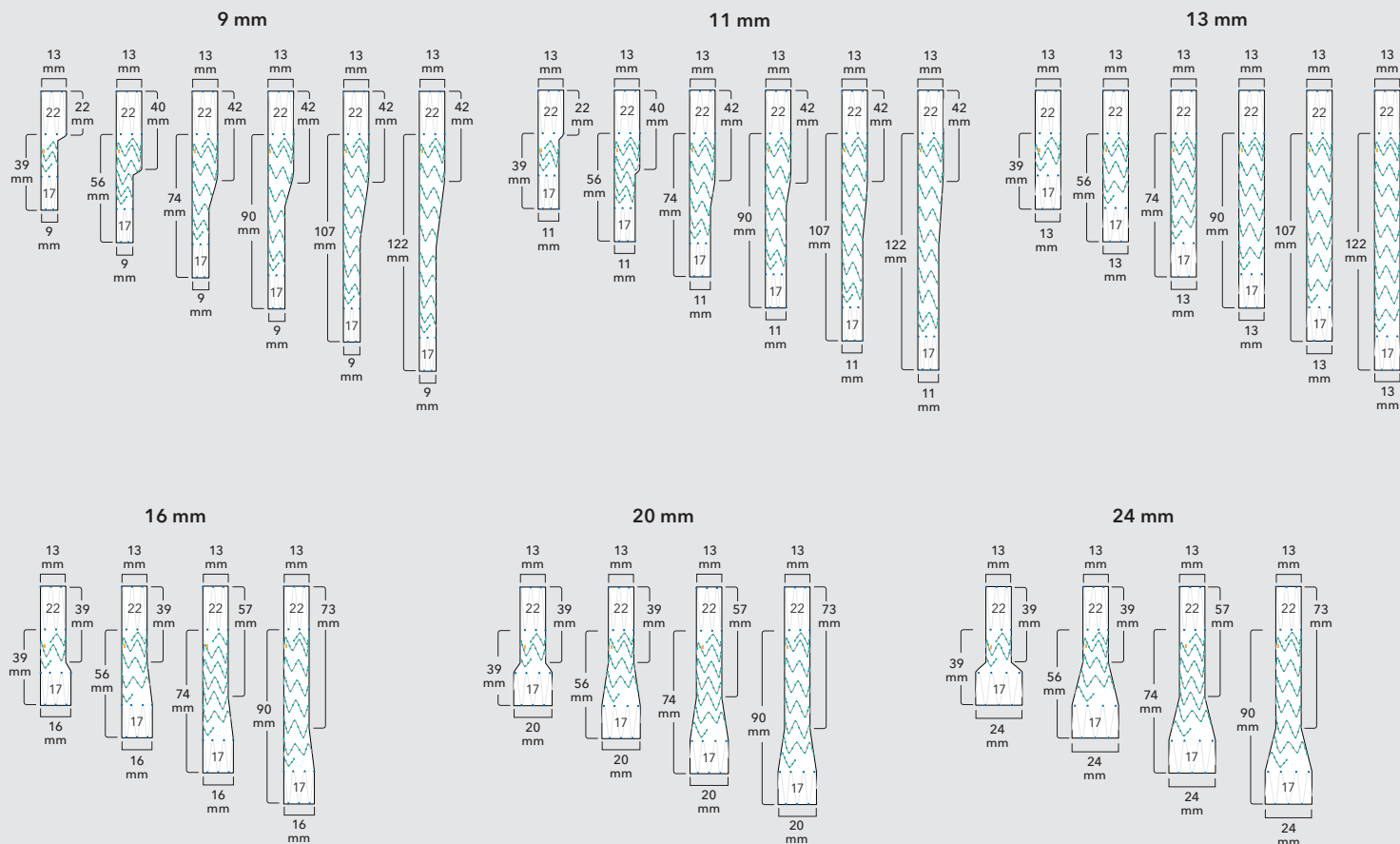


Note: Black numbers on stents indicate stent lengths in millimeters.

Z-Trak® introduction system for iliac legs

9-16 mm grafts: 14 Fr (4.7 mm ID/5.4 mm OD)
20-24 mm grafts: 16 Fr (5.3 mm ID/6 mm OD)

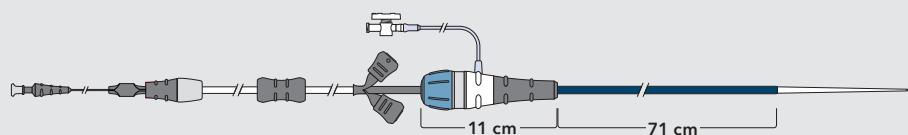


Zenith Spiral-Z AAA iliac leg graft - Component ordering information

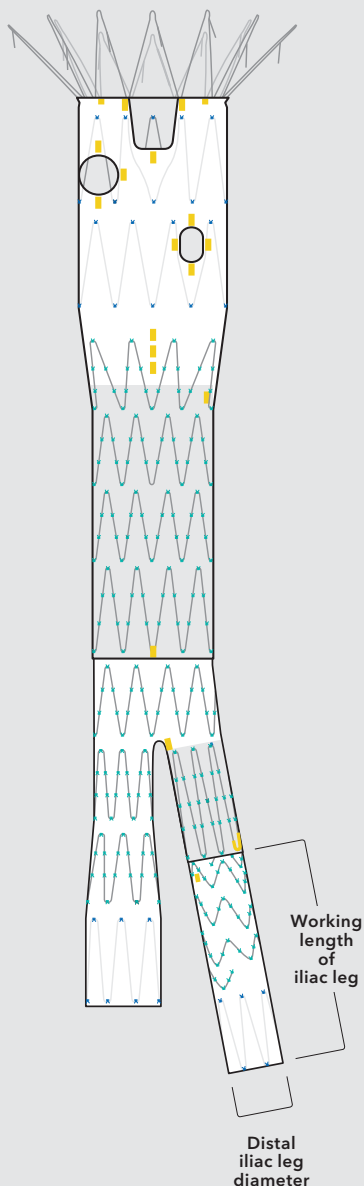

Order Number	Reference Part Number	Distal Iliac Leg Diameter mm	Working Length of Iliac Leg mm	Introducer Sheath ID/OD Fr (mm)/mm
9 mm Diameter				
G55218	ZSLE-9-39-ZT	9	39	14 (4.7)/5.4
G55219	ZSLE-9-56-ZT	9	56	14 (4.7)/5.4
G55220	ZSLE-9-74-ZT	9	74	14 (4.7)/5.4
G55221	ZSLE-9-90-ZT	9	90	14 (4.7)/5.4
G55222	ZSLE-9-107-ZT	9	107	14 (4.7)/5.4
G55223	ZSLE-9-122-ZT	9	122	14 (4.7)/5.4
11 mm Diameter				
G55224	ZSLE-11-39-ZT	11	39	14 (4.7)/5.4
G55225	ZSLE-11-56-ZT	11	56	14 (4.7)/5.4
G55226	ZSLE-11-74-ZT	11	74	14 (4.7)/5.4
G55227	ZSLE-11-90-ZT	11	90	14 (4.7)/5.4
G55228	ZSLE-11-107-ZT	11	107	14 (4.7)/5.4
G55229	ZSLE-11-122-ZT	11	122	14 (4.7)/5.4
13 mm Diameter				
G55230	ZSLE-13-39-ZT	13	39	14 (4.7)/5.4
G55231	ZSLE-13-56-ZT	13	56	14 (4.7)/5.4
G55232	ZSLE-13-74-ZT	13	74	14 (4.7)/5.4
G55233	ZSLE-13-90-ZT	13	90	14 (4.7)/5.4
G55234	ZSLE-13-107-ZT	13	107	14 (4.7)/5.4
G55235	ZSLE-13-122-ZT	13	122	14 (4.7)/5.4
16 mm Diameter				
G55236	ZSLE-16-39-ZT	16	39	14 (4.7)/5.4
G55237	ZSLE-16-56-ZT	16	56	14 (4.7)/5.4
G55238	ZSLE-16-74-ZT	16	74	14 (4.7)/5.4
G55239	ZSLE-16-90-ZT	16	90	14 (4.7)/5.4
20 mm Diameter				
G55240	ZSLE-20-39-ZT	20	39	16 (5.3)/6
G55241	ZSLE-20-56-ZT	20	56	16 (5.3)/6
G55242	ZSLE-20-74-ZT	20	74	16 (5.3)/6
G55243	ZSLE-20-90-ZT	20	90	16 (5.3)/6
24 mm Diameter				
G55244	ZSLE-24-39-ZT	24	39	16 (5.3)/6
G55245	ZSLE-24-56-ZT	24	56	16 (5.3)/6
G55246	ZSLE-24-74-ZT	24	74	16 (5.3)/6
G55247	ZSLE-24-90-ZT	24	90	16 (5.3)/6

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.






Zenith alignment stent - Component ordering information

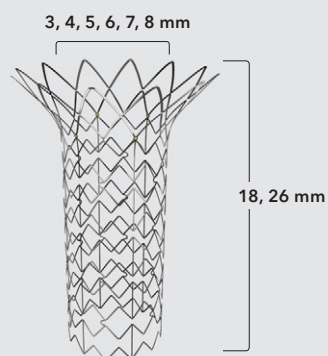

Order Number	Reference Part Number	Diameter mm	Length mm	Minimum Introducer Sheath ID Fr	Minimum Guiding Catheter OD Fr
G49276	ZFS-80-3-18	3	18	7	8
G49277	ZFS-80-3-26	3	26	7	8
G49278	ZFS-80-4-18	4	18	7	8
G49279	ZFS-80-4-26	4	26	7	8
G49280	ZFS-80-5-18	5	18	7	8
G49281	ZFS-80-5-26	5	26	7	8
G49282	ZFS-80-6-18	6	18	7	8
G49283	ZFS-80-6-26	6	26	7	8
G49284	ZFS-80-7-18	7	18	7	8
G49285	ZFS-80-7-26	7	26	7	8
G49286	ZFS-80-8-18	8	18	7	8
G49287	ZFS-80-8-26	8	26	7	8

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



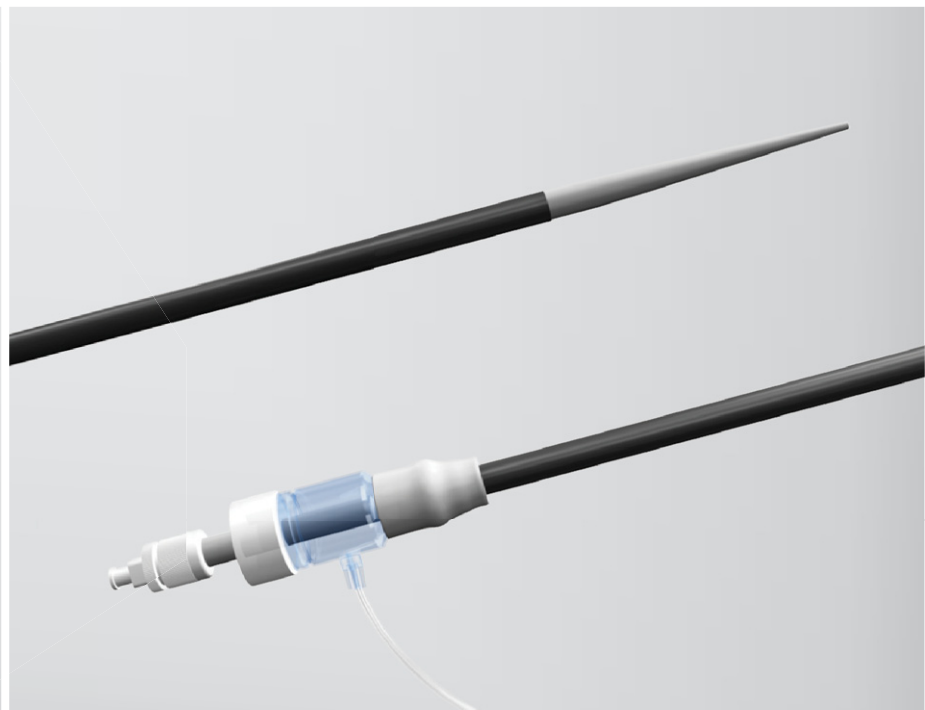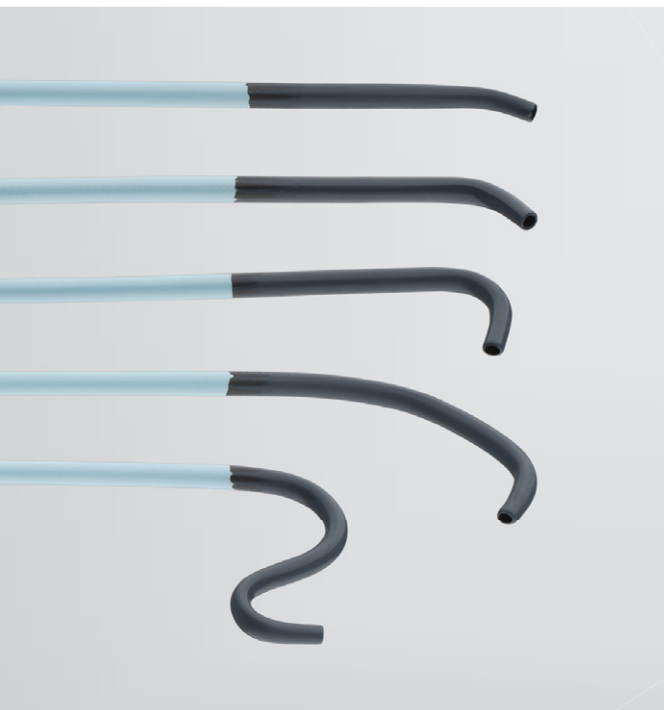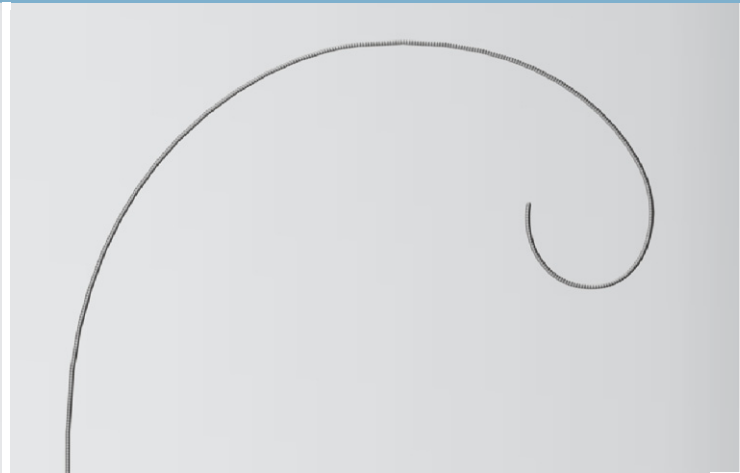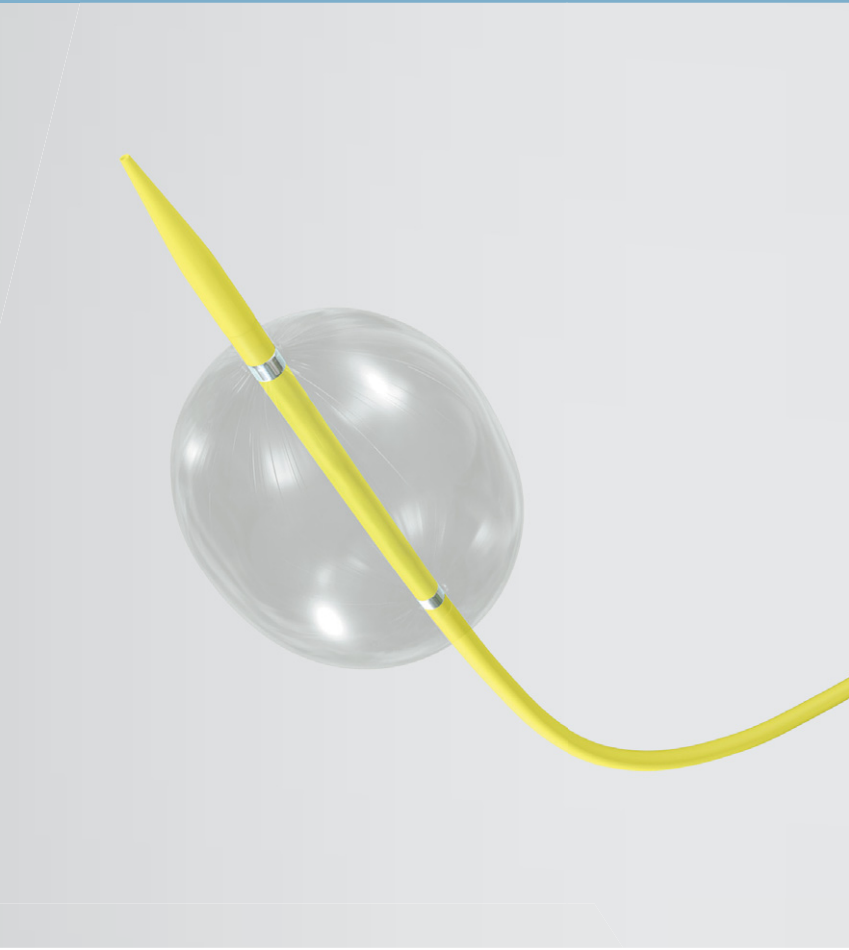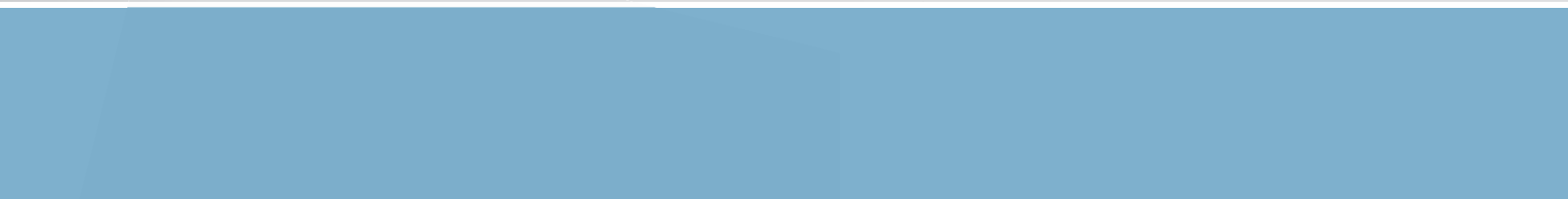

Select case-specific EVAR accessories.

ACCESSORIES FOR FENESTRATED ENDOVASCULAR PROCEDURES

We have the accessories you need for your fenestrated EVAR cases, including access needles, dilators, introducers, wire guides, and a variety of catheters.

Order Number	Reference Part Number	Description	Quantity
Access Needle			
G04876	SDN-19UT-7.0	Vascular Access Needle	1
Balloon			
G53024	CODA-2-10.0-35-120-32	Coda® Balloon Catheter	1
Catheters			
G13795	HNBR5.0-35-65-P-NS-VANSCHIE1	Van Schie Beacon® Tip Seeking Catheter	1
G13794	HNBR5.0-35-65-P-NS-VANSCHIE2	Van Schie Beacon Tip Seeking Catheter	1
G13792	HNBR5.0-35-65-P-NS-VANSCHIE3	Van Schie Beacon Tip Seeking Catheter	1
G13783	HNBR5.0-35-65-P-NS-VANSCHIE4	Van Schie Beacon Tip Seeking Catheter	1
G13793	HNBR5.0-35-65-P-NS-VANSCHIE5	Van Schie Beacon Tip Seeking Catheter	1
G08441	HNBR5.0-35-65-P-NS-C1	Torcon NB® Advantage Beacon Tip Catheter, C1	5
G08442	HNBR5.0-35-65-P-NS-C2	Torcon NB Advantage Beacon Tip Catheter, C2	5
G08444	HNBR5.0-35-65-P-NS-RIM	Torcon NB Advantage Beacon Tip Catheter, RIM	5
G09469	HNBR5.0-38-65-P-NS-KMP	Torcon NB Advantage Beacon Tip Catheter, KMP	5
G08677	HNBR5.0-38-65-P-NS-SIM1	Torcon NB Advantage Beacon Tip Catheter, SIM1	5
G08625	HNBR5.0-38-65-P-NS-SIM2	Torcon NB Advantage Beacon Tip Catheter, SIM2	5
G11189	HNBR5.0-38-80-P-NS-VS1	Torcon NB Advantage Beacon Tip Angiographic Catheter	5
G12007	SCBR5.0-38-100-P-NS-VERT	Slip-Cath® Beacon Tip Catheter, VERT	5
G12008	SCBR5.0-38-100-P-NS-C2	Slip-Cath Beacon Tip Catheter, C2	5
G09411	HNR5.0-35-70-P-10S-0	Beacon Tip Royal Flush® Plus Straight Catheter	5
G11267	HNR5.0-38-100-P-10S-0	Beacon Tip Royal Flush Plus Straight Catheter	5
G09059	HNR5.0-35-100-P-10S-PIG	Beacon Tip Royal Flush Plus Pigtail Catheter	5
G31216	NR5.0-35-100-P-10S-PIG-CSC-20	Aurous™ Beacon Tip Pigtail Sizing Catheter	5
G52546	CXI-4.0-35-90-P-NS-0	CXI Support Catheter	1
Dilator Sets			
G52833	JCDS-14-EDS-HC	Endovascular Dilator Set	1
G52834	JCDS-1618-EDS-HC	Endovascular Dilator Set	2
G52835	JCDS-2022-EDS-HC	Endovascular Dilator Set	2
G52836	JCDS-2426-EDS-HC	Endovascular Dilator Set	2

Continued on page 18

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Accessories for Fenestrated Endovascular Procedures - Ordering information

Continued from page 17

Order Number	Reference Part Number	Description	Quantity
Introducers			
G07338	RCF-7.0-35-J	Check-Flo Performer® Introducer	1
G56236	KCFW-6.0-35-55-RB-HFANL1-HC	Flexor® Ansel Check-Flo® Introducer with High-Flex Dilator	1
G49043	KCFW-7.0-35-55-RB-HFANL1-HC	Flexor Ansel Check-Flo Introducer with High-Flex Dilator	1
G49045	KCFW-8.0-35-55-RB-HFANL1-HC	Flexor Ansel Check-Flo Introducer with High-Flex Dilator	1
G28999	XVCFW-20.0-35-25	Extra-Large Check-Flo Performer Introducer	1
G12831	XVCFW-22.0-35-25	Extra-Large Check-Flo Performer Introducer	1
G12896	XVCFW-24.0-35-25	Extra-Large Check-Flo Performer Introducer	1
Wire Guides			
G00691	TSFB-35-145	Bentson Straight Fixed Core Wire Guide	1
G01063	TSFB-35-260	Bentson Straight Fixed Core Wire Guide	1
G45353	TSCMG-35-260-7-LES	Lunderquist® Extra-Stiff Curved Exchange Support Wire Guide	5
G45208	TSCMG-35-260-LESDC	Lunderquist Extra-Stiff Double Curved Exchange Support Wire Guide	5
G07518	RPC-35-180	Roadrunner® PC Wire Guide, The Firm	5
G09105	RPC-35-260	Roadrunner PC Wire Guide, Exchange	1
G01253	THSCF-35-260-1.5-ROSEN	Rosen Heavy Duty Wire Guide	1
G20320	THSCF-35-260-3-AUS2	Amplatz Stiff Curved Exchange Wire Guide	1

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.



Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
Distributors: +353 61239240, ssc.distributors@cookmedical.com
Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
Finland: +358 972519996, fi.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858769468, se.orders@cookmedical.com
Switzerland - French: +41 448009609, fr.orders@cookmedical.com
Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
Switzerland - German: +41 448009609, de.orders@cookmedical.com
United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com/web/aortic-intervention/

Americas: EDI - www.cookmedical.com/edi.do
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
E-mail: orders@cookmedical.com

Australia:
Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
E-mail: cau.custserv@cookmedical.com