




ICD Lead with single central lumen coil and two cable conductors.



Remove proximal connector using Lead Clippers and expose central lumen coil and cable conductors.

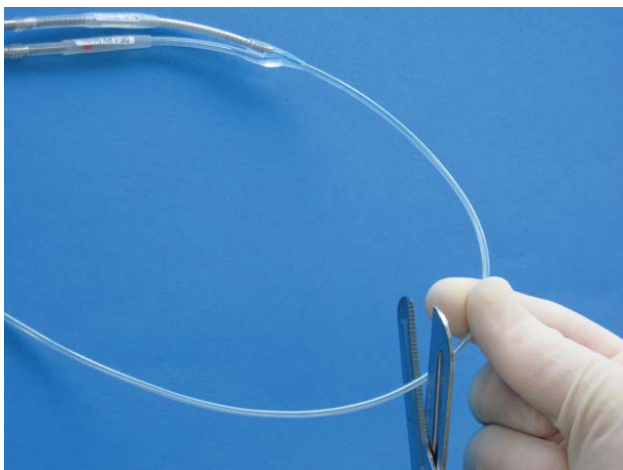


Using scalpel or other device carefully remove lead's outer insulation.

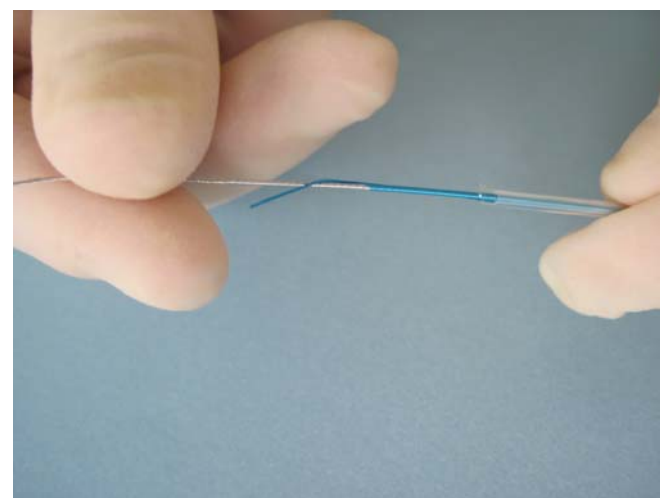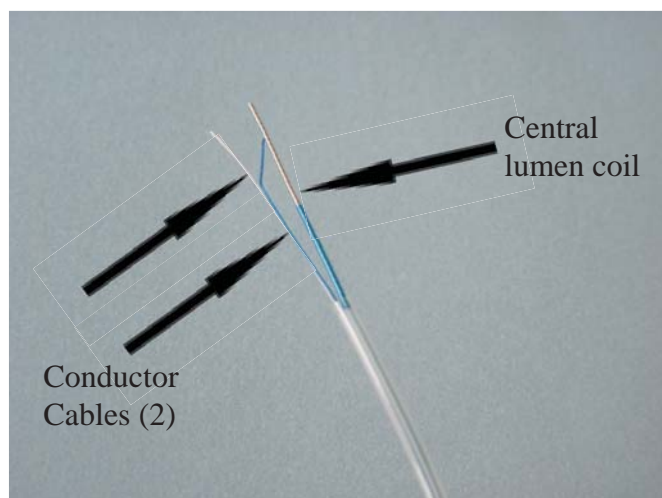

Insert Liberator® Locking Stylet into central lumen coil. Lock in distal end of coil.




Tie suture firmly around entire lead body (insulation, coil and cables) at a location 2-4 cm distal to proximal end of insulation.

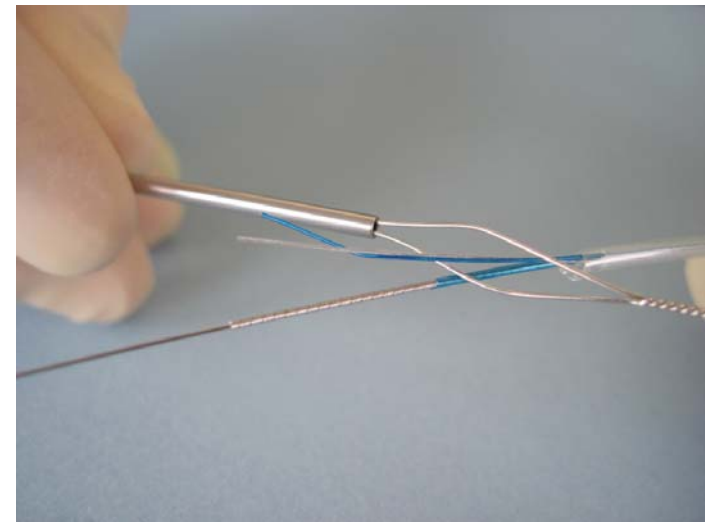

Place one or both conductor cables through loop of Bulldog™ Lead Extender.



Advanced actuator tube of Bulldog Lead Extender forward until cable(s) are locked.

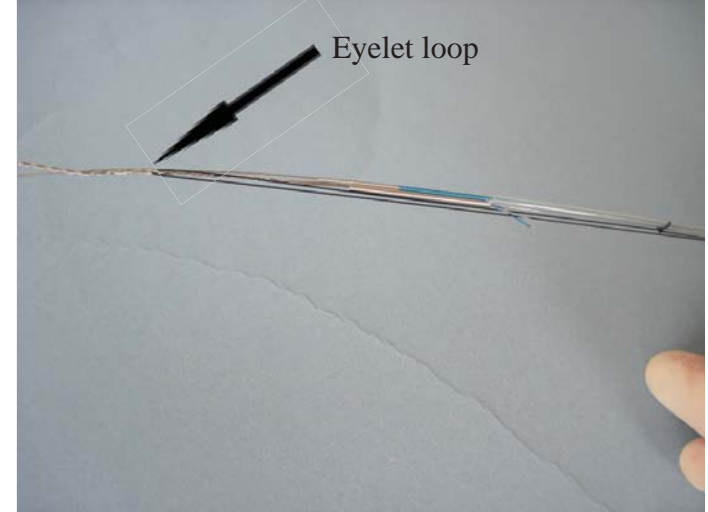

(Optional) Tie end of suture to eyelet loop in Bulldog's twisted wire handle or to the loop on the actuator tube of the Liberator Locking Stylet.