



VersaTube™

TAPERED TRACHEOSTOMY TUBE

Care and Maintenance of VersaTube and Inner Cannula (IC)

- Current facility protocol for tracheostomy tube care and maintenance will also apply to VersaTube.
- VersaTube sizes and the corresponding IC are color coded for simple and efficient identification.

VersaTube IC Changes

- Follow your facility's current protocol for IC changes. To simplify this process, ensure box of appropriate-size ICs are readily available in the patient's room.
- Two small round bumps located on the top and bottom of the IC help to secure it tightly inside the tube, while a ring-pull feature at the proximal end of the IC contributes to ease of removal.

Important Information

- **VersaTube's IC should be used with VersaTube tracheostomy tube only.**
The tube is not compatible with any other manufacturer's inner cannula.
- The VersaTube Tapered Tracheostomy Tube is designed for use with percutaneous dilatational tracheostomy and balloon-assisted tracheostomy devices.
- VersaTube's cuff pressure should never exceed 25 mm Hg.
- The VersaTube Tapered Tracheostomy Tube is made of a soft PVC-blend material (tube, flange and cuff) to maximize patient comfort.
- VersaTube's 15 mm connector allows for ventilation with or without the inner cannula.

Ordering Information

Order Number	Reference Part Number	Tracheostomy Tube ID mm	Tracheostomy Tube OD mm	Tracheostomy Tube Length mm	Angle degrees	Cuff Resting Diameter mm
VersaTube Tapered Tracheostomy Tubes						
G54914	C-VT-7	7	10	78	96	25
G54915	C-VT-8	8	11	86	96	28
G54916	C-VT-9	9	12	98	97	30
Order Number	Reference Part Number	Inner Cannula ID mm	Corresponding VersaTube ISO size	VersaTube Tracheostomy Tube Color Code		

VersaTube Inner Cannulas

G54918	C-VTIC-7	6	7.0	green
G54919	C-VTIC-8	7	8.0	white
G54920	C-VTIC-9	8	9.0	blue

Note: VersaTube may have a smaller outer diameter and/or may be longer than the previous tracheostomy tube used at your facility.

Passey-Muir is a trademark of Passey-Muir Inc.

VersaTube Size Conversions

Current Tube*

VersaTube

		ID with IC mm	ID without IC mm	OD mm	Length mm	Corresponding VersaTube ISO size	ID with IC mm	ID without IC mm	OD mm	Length mm
Shiley®	PERC 6	6.4	n/a	10.8	74	7.0	6	7	10	78
	PERC 8	7.6	n/a	12.2	79	8.0	7	8	11	86
	Tracheosoft® 7.0	7	n/a	12.3	100	7.0	6	7	10	78
	Tracheosoft 8.0	8	n/a	13.3	105	8.0	7	8	11	86
Portex®	Blue Line Ultra® 7.0	5.5	7	10.5	70	7.0	6	7	10	78
	Blue Line Ultra 8.0	6.5	8	11.9	75.5	8.0	7	8	11	86
	Blue Line Ultra 9.0	7.5	9	13.3	81	9.0	8	9	12	98
	Per-fit™ 7.0	6	7	9.6	82	7.0	6	7	10	78
	Per-fit 8.0	7	8	10.9	86	8.0	7	8	11	86
	Per-fit 9.0	8	9	12.3	93	9.0	8	9	12	98
Tracoe®	Twist 7.0	7	n/a	10.4	80.5	7.0	6	7	10	78
	Twist 8.0	8	n/a	11.4	83	8.0	7	8	11	86
	Twist 9.0	9	n/a	12.5	88.5	9.0	8	9	12	98

*Shiley uses Jackson sizing system; all others use ISO.

Blue Line Ultra, Per-fit and Portex are trademarks
of Smiths Medical International.

Shiley and Tracheosoft are registered trademarks
of Mallinckrodt Inc.

Tracoe is a registered trademark of Tracoe Medical GmbH.

Contact your local Cook Medical
representative with questions.



Customer Service Centers

AORTIC INTERVENTION

CRITICAL CARE

ENDOSCOPY

INTERVENTIONAL RADIOLOGY

LEAD MANAGEMENT

PERIPHERAL INTERVENTION

SURGERY

UROLOGY

WOMEN'S HEALTH

EMEA: EDI - www.cookmedical.com/edi.do
Distributors: +353 61239240, ssc.distributors@cookmedical.com
Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Poland: +48 223060159, pl.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858769468, se.orders@cookmedical.com
Switzerland - French: +41 448009609, fr.orders@cookmedical.com
Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
Switzerland - German: +41 448009609, de.orders@cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
E-mail: orders@cookmedical.com

Australia:
Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
E-mail: caucustserv@cookmedical.com

www.cookmedical.com

