



Manage postpartum
hemorrhage simply
and effectively.

Cover illustration by Lisa Clark.

Bakri®

POSTPARTUM BALLOON WITH RAPID INSTILLATION COMPONENTS



www.cookmedical.com

A simple solution for postpartum hemorrhage




The Bakri Postpartum Balloon with Rapid Instillation Components is an effective way to treat postpartum hemorrhage (PPH) after a cesarean section or vaginal delivery when conservative management of uterine bleeding is warranted.

Managing postpartum hemorrhage

Balloon tamponade is an effective adjunct in the treatment of severe postpartum hemorrhage secondary to uterine atony, especially when medical therapy fails.^{1, 2}


Treatment modalities³

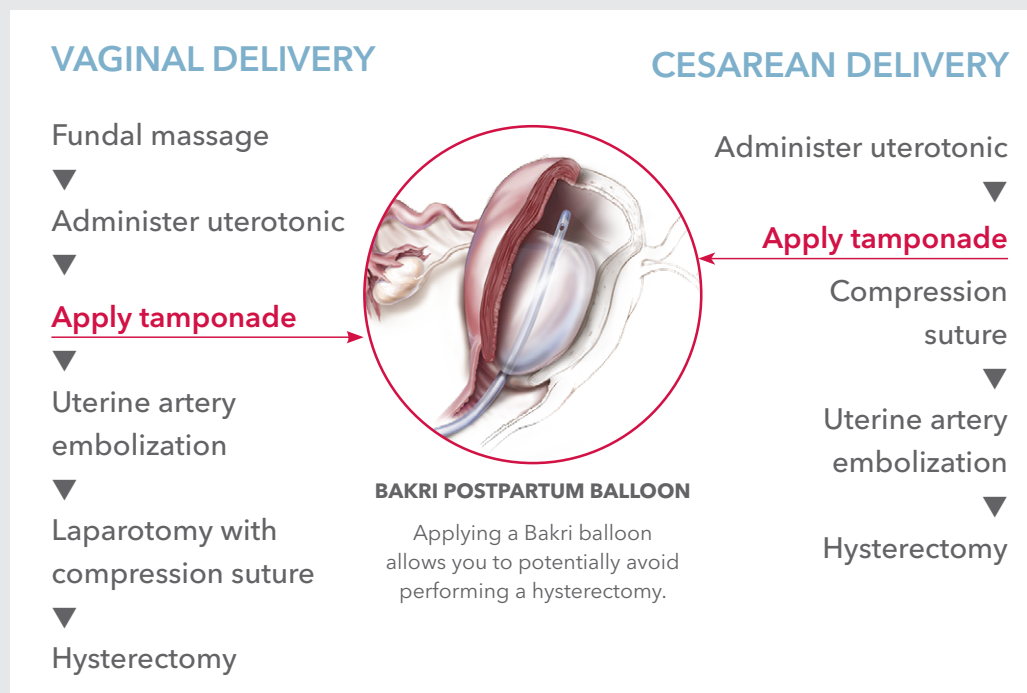


A first-line treatment for PPH that is resistant to uterotonic agents.⁴

When inflated with sterile liquid, the Bakri Postpartum Balloon with Rapid Instillation Components conforms to the shape of the uterus and may provide immediate management of postpartum hemorrhage.

- The silicone construction eliminates the problem of patient sensitivity to latex.
- The balloon is easy to position and remove.
- The balloon is designed to control bleeding simply.
- The drainage lumen allows blood loss monitoring.
- The rapid instillation component enables timely instillation of saline into the balloon.

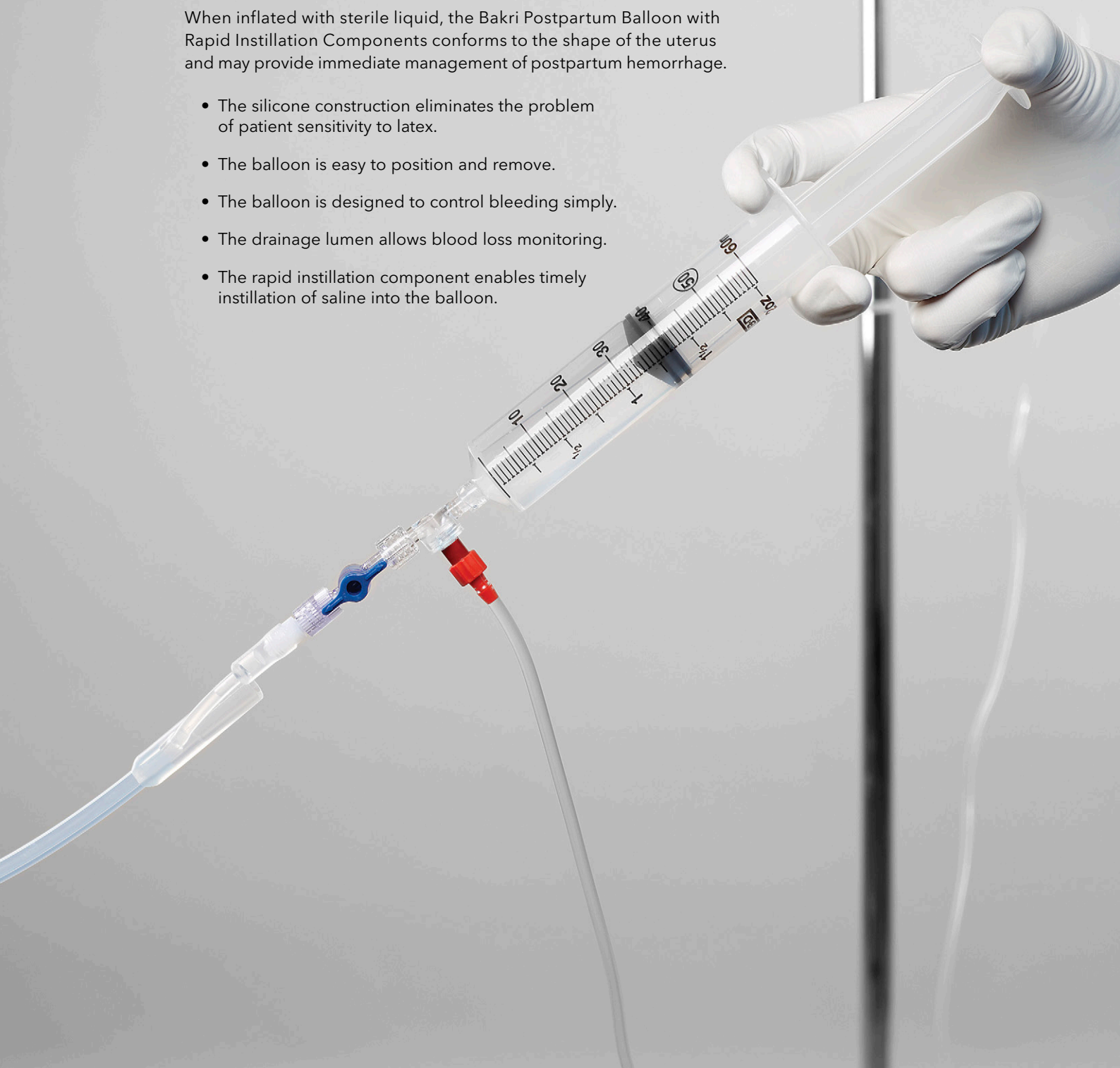

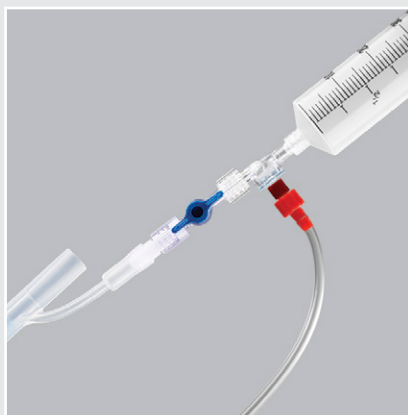

Bakri®

POSTPARTUM BALLOON WITH RAPID INSTILLATION COMPONENTS

The incorporation of a rapid instillation component with the Bakri Postpartum Balloon enables timely instillation of saline into the balloon when conservative management of uterine bleeding is warranted. The set includes a 60 mL syringe. Please refer to the Instructions for Use for complete information on product usage, proper indications, and contraindications.

Order Number	Reference Part Number	Catheter Fr	Length cm	Balloon Volume mL
Bakri Postpartum Balloon with Rapid Instillation Components				
G24237	J-SOSR-100500	24	54	500

Includes a dual check valve and 180 cm long IV tubing with attached bag spike.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

References

1. American College of Obstetricians and Gynecologists, ACOG Practice Bulletin: clinical management guidelines for obstetrician-gynecologists number 76, October 2006: postpartum hemorrhage. *Obstet Gynecol.* 2006;108(4):1039-1047.
2. Dabelea V, Schultze PM, McDuffie RS Jr. Intrauterine balloon tamponade in the management of postpartum hemorrhage. *Am J Perinatol.* 2007;24(6):359-364.
3. California Maternal Quality Care Collaborative. Obstetric hemorrhage care summary: table chart format, version 1.4. Published 2010. Accessed January 20, 2014.
4. Kaya B, Tuten A, Daglar K, et al. Balloon tamponade for the management of postpartum uterine hemorrhage. *J Perinat Med.* 2014;42(6):745-753.

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
Distributors: +353 61239240, ssc.distributors@cookmedical.com
Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
Finland: +358 972519996, fi.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com
Norway: +47 23162968, no.orders@cookmedical.com
Spain: +34 912702691, es.orders@cookmedical.com
Sweden: +46 858769468, se.orders@cookmedical.com
Switzerland - French: +41 448009609, fr.orders@cookmedical.com
Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
Switzerland - German: +41 448009609, de.orders@cookmedical.com
United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
E-mail: orders@cookmedical.com

Australia:
Phone: +61 734346000, 1800777222, Fax: +61 734346001, 1800077283
E-mail: cau.custserv@cookmedical.com

