
## More control, less stress.



## Evolution Esophageal

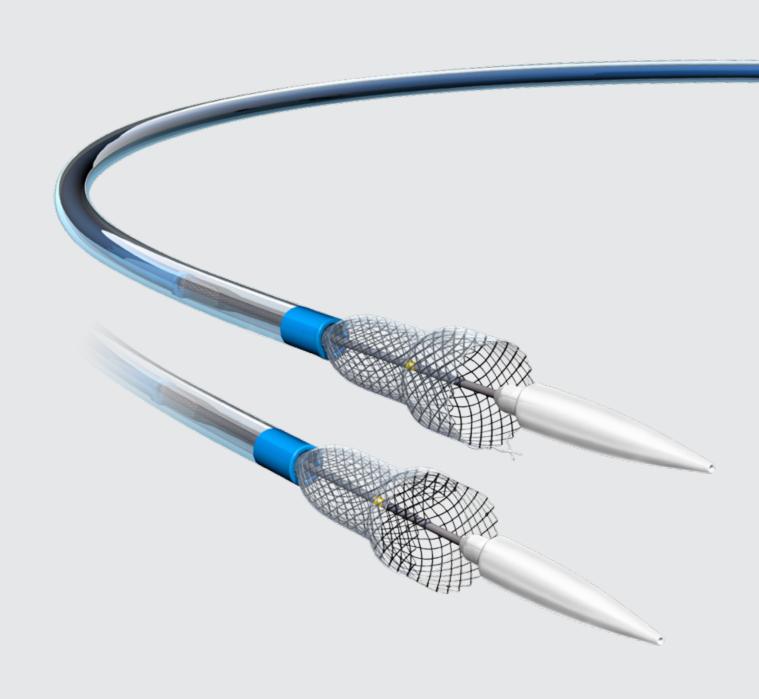


# More control, less stress.



### **Evolutionary Design**

Encased inside and out with silicone, Evolution is designed to reduce ingrowth and decrease food bolus impaction. Proximal and distal flanges, either covered or uncovered, secure the stent position and lessen the risk of migration. A proximal lasso loop on the partially covered stent and dual lasso loops on both the proximal and distal ends of the fully covered stent gives the freedom to reposition the stent immediately after placement.

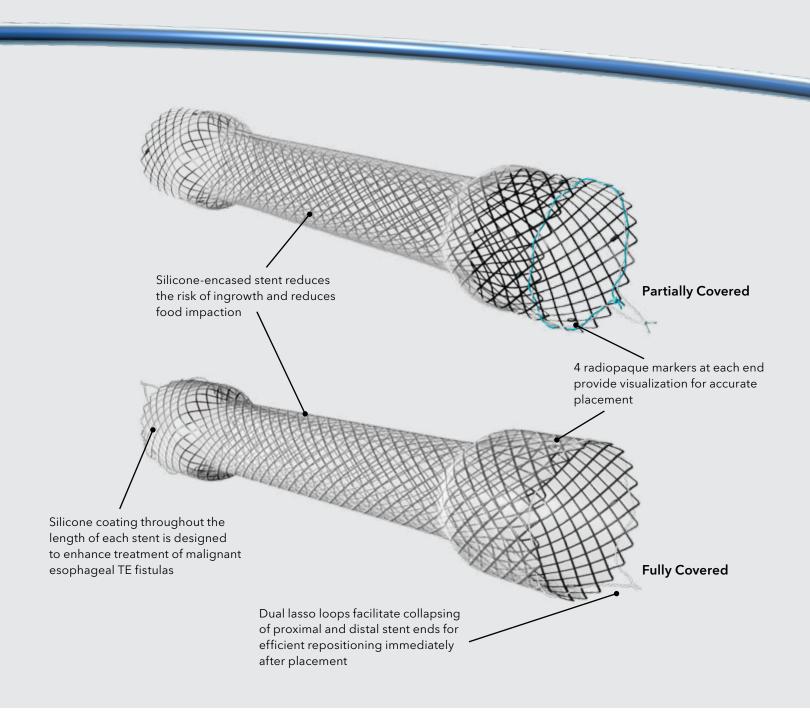


### **Evolutionary Control**

Each squeeze of the trigger, gives you precise control over stent deployment and recapturability. A simple switch of the Directional button allows you to move between release and recapture with ease. There is even a "point of no return" reference mark to alert you when stent capture is no longer possible.



### **Evolutionary Cover**

To give you the widest range of treatment options, Evolution Esophageal stents come partially or fully covered. And they all deploy smoothly and confidently thanks to the stress-reducing, controlled release delivery system.

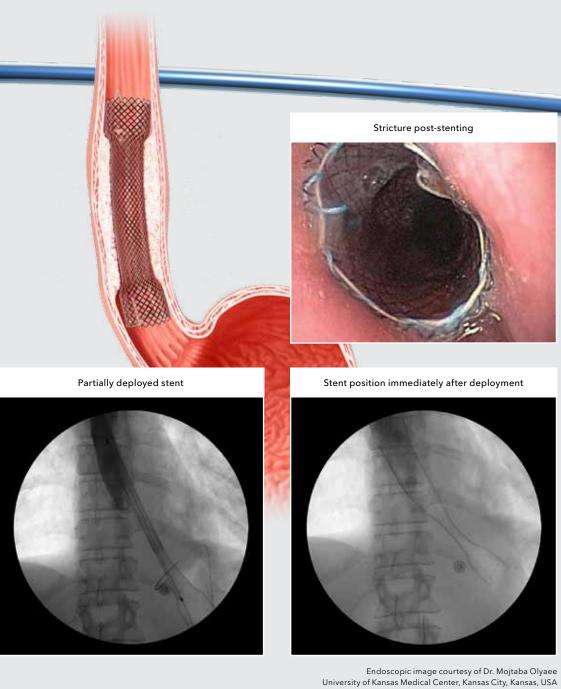

Fluoroscopic images courtesy of Dr. Marc Giovannini

#### **Product Specifications**

#### **Fully Covered**

Used to maintain patency of malignant esophageal strictures and/or to seal tracheoesophageal fistulas.

Supplied sterile and is disposable - intended for single use only.

Order Number	Reference Part Number	Body Diameter mm	Flange Diameter mm	Stent Length cm	Delivery System Diameter mm
G51181	EVO-FC-18-23-8-E	18	23	8	8
G51182	EVO-FC-18-23-10-E	18	23	10	8
G51183	EVO-FC-18-23-12-E	18	23	12	8
G52225	EVO-FC-20-25-8-E	20	25	8	8
G52227	EVO-FC-20-25-10-E	20	25	10	8
G52226	EVO-FC-20-25-12-E	20	25	12	8

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

#### Partially Covered

Used to maintain patency of malignant esophageal strictures and/or to seal tracheoesophageal fistulas.

Supplied sterile and is disposable - intended for single use only.

Order Number	Reference Part Number	Body Diameter mm	Flange Diameter mm	Stent Length cm	Delivery System Diameter mm
G48030	EVO-20-25-8-E	20	25	8	8
G48031	EVO-20-25-10-E	20	25	10	8
G48032	EVO-20-25-12.5-E	20	25	12.5	8
G48033	EVO-20-25-15-E	20	25	15	8

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

#### **Customer Service**

EMEA: EDI - www.cookmedical.com/edi.do

Distributors: +353 61239240, ssc.distributors@cookmedical.com

Austria: +43 179567121, oe.orders@cookmedical.com
Belgium: +32 27001633, be.orders@cookmedical.com
Denmark: +45 38487607, da.orders@cookmedical.com
Finland: +358 972519996, fi.orders@cookmedical.com
France: +33 171230269, fr.orders@cookmedical.com
Germany: +49 6950072804, de.orders@cookmedical.com
Hungary: +36 17779199, hu.orders@cookmedical.com
Ireland: +353 61239252, ie.orders@cookmedical.com
Italy: +39 0269682853, it.orders@cookmedical.com
Netherlands: +31 202013367, nl.orders@cookmedical.com

Norway: +47 23162968, no.orders@cookmedical.com Spain: +48 212702691, es.orders@cookmedical.com Sweden: +46 858769468, se.orders@cookmedical.com

Switzerland - French: +41 448009609, fr.orders@cookmedical.com Switzerland - Italian: +41 448009609, it.orders@cookmedical.com Switzerland - German: +41 448009609, de.orders@cookmedical.com United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

**Americas:** EDI - www.cookmedical.com/edi.do Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335

E-mail: orders@cookmedical.com

Australia:

Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283

E-mail: cau.custserv@cookmedical.com

