


# Evolution<sup>®</sup> Colonic

CONTROLLED-RELEASE STENT - UNCOVERED

## Features



## Step 1

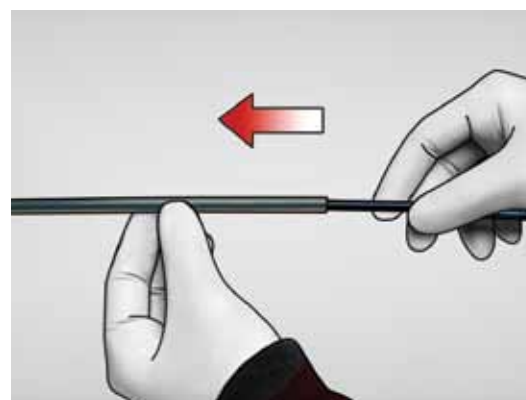

Remove protective tubing from stent.


## Step 2



Introduce delivery system in short increments over wire guide, into accessory channel, until it is endoscopically visualized exiting scope.

**Note:** Minimum accessory channel 3.7 mm

## Step 3a

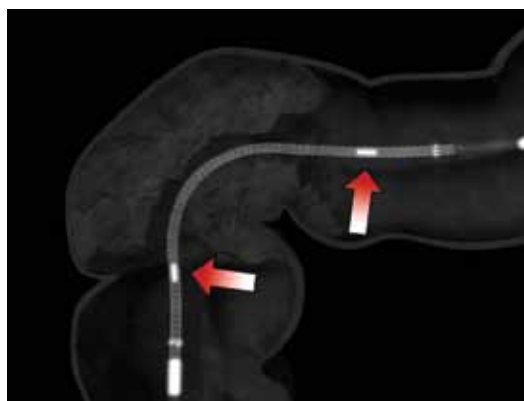


Fluoroscopically position radiopaque markers on inner catheter beyond extremities of stricture to be crossed.

## Step 3b



**Note:** Yellow marker on the delivery system located at proximal end of stent can also serve as a endoscopic/fluoroscopic reference for positioning of the proximal (nearest user) end of the stent relative to the stricture.

## Step 4

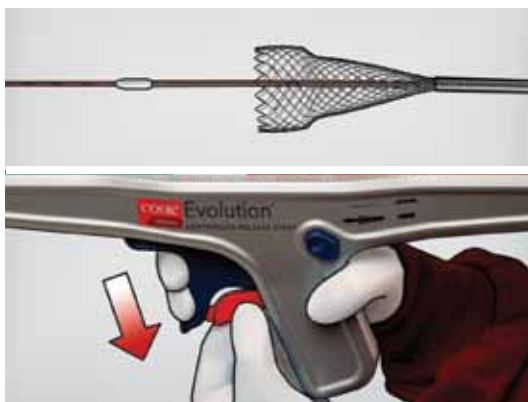

Confirm desired stent position fluoroscopically and deploy stent by removing red safety guard from handle. Continue deploying stent by squeezing trigger.


## Step 5a



If repositioning is required during deployment, it is possible to recapture stent.

**Note:** It is not possible to recapture stent after passing point-of-no-return, indicated when the stent deployment indicator on top of introducer has passed point-of-no-return indicator on handle. The stent can be captured a maximum of 5 times.

## Step 5b

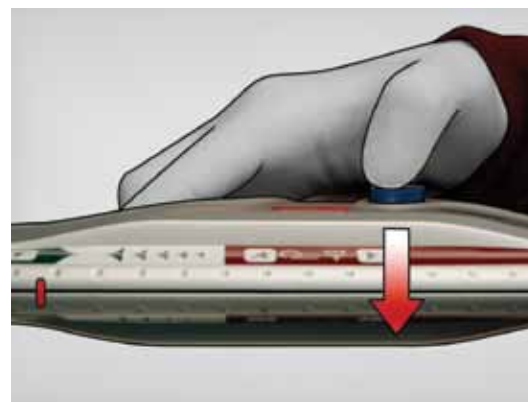

To recapture stent, push directional button on side of delivery system to opposite side.


**Note:** Hold thumb on button when squeezing trigger for first time to recapture. Continue squeezing trigger as required to recapture stent by desired amount.

## Step 6




To resume deployment, push directional button to opposite side again and hold button for first stroke while squeezing trigger.

## Step 7

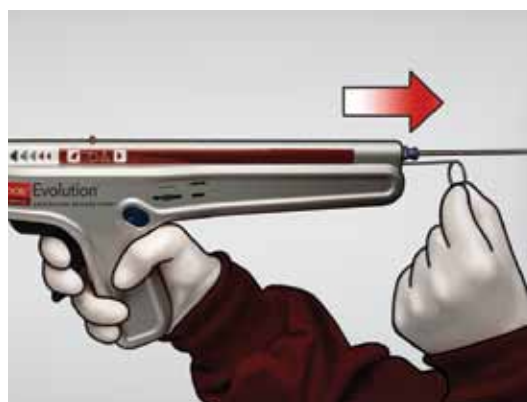

When stent point-of-no-return has been passed, pull safety wire out of delivery handle near wire guide port. Continue deploying stent by squeezing trigger.

## Step 8

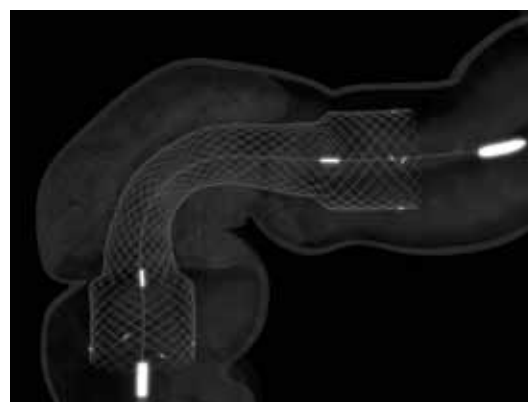

After deployment, fluoroscopically confirm stent expansion. Once expansion is confirmed, introduction system can be safely removed.

Refer to current instructions for detailed system use.

### Customer Service

**EMEA:** EDI - [www.cookmedical.com/edi.do](http://www.cookmedical.com/edi.do)  
Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
Belgium: +32 27001633, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
Norway: +47 23162968, [no.orders@cookmedical.com](mailto:no.orders@cookmedical.com)  
Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

[www.cookmedical.com](http://www.cookmedical.com)

**Americas:** EDI - [www.cookmedical.com/edi.do](http://www.cookmedical.com/edi.do)  
Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
E-mail: [orders@cookmedical.com](mailto:orders@cookmedical.com)

**Australia:**  
Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283  
E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)

