

Efficiently treat bulging pancreatic pseudocysts.



Cystotome[®]

CYSTOENTEROSTOMY NEEDLE KNIFE

With an electrosurgical needle knife and a diathermic ring, the Cook Medical Cystotome is a combination device that allows efficient treatment for bulging pancreatic pseudocysts. The wire-guided Cystotome also maintains position throughout therapeutic treatment for easier and faster endoscopic cystoenterostomy.



www.cookmedical.com



Cystotome®

CYSTOENTEROSTOMY NEEDLE KNIFE

Used to electrosurgically puncture a hole in the transgastric or transduodenal wall and into a pancreatic pseudocyst, when it is visibly bulging into the gastrointestinal tract.

Supplied sterile and is disposable – intended for single use only.

- Make initial incision with .038" needle knife in a 5 Fr catheter
- Immediately enlarge incision with follow-on cauterizing diathermic ring and 10 Fr outer catheter
- Accepts .035" wire guide to maintain position after first incision and facilitate placement of a stent or a drainage kit
- Combining a needle knife and diathermic ring eliminates need for instrument change, reducing procedure time
- Multiple integrated components – needle knife, diathermic ring, and wire-guide compatibility – are a platform for expeditious delivery of therapeutic treatment

Order Number	Reference Part Number	Outer Catheter		Inner Catheter		Diathermic Ring	Color Connection
		French Size Fr	Length cm	French Size Fr	Length cm	French Size Fr	
G30550	CST-10	10	165	5	190	10	Purple

Minimum accessory channel 3.7 mm. *Recommend Tracer Metro Direct wire guide, sold separately. Active cord available separately.

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do

Distributors: +353 61239240, ssc.distributors@cookmedical.com

Austria: +43 179567121, oe.orders@cookmedical.com

Belgium: +32 27001633, be.orders@cookmedical.com

Denmark: +45 38487607, da.orders@cookmedical.com

Finland: +358 97251996, fi.orders@cookmedical.com

France: +33 171230269, fr.orders@cookmedical.com

Germany: +49 6950072804, de.orders@cookmedical.com

Hungary: +36 17779199, hu.orders@cookmedical.com

Ireland: +353 61239252, ie.orders@cookmedical.com

Italy: +39 0269682853, it.orders@cookmedical.com

Netherlands: +31 202013367, nl.orders@cookmedical.com

Norway: +47 23162968, no.orders@cookmedical.com

Spain: +34 912702691, es.orders@cookmedical.com

Sweden: +46 858769468, se.orders@cookmedical.com

Switzerland - French: +41 448009609, fr.orders@cookmedical.com

Switzerland - Italian: +41 448009609, it.orders@cookmedical.com

Switzerland - German: +41 448009609, de.orders@cookmedical.com

United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do

Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335

E-mail: orders@cookmedical.com

Australia:

Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283

E-mail: cau.custserv@cookmedical.com

